



This declaration of consent must be completed by a parent or legal guardian.

Declaration of consent

First name: _____
Last name: _____
Street: _____
Postcode: _____
City: _____
Country: _____
E-mail: _____
Tel: _____

I,....., hereby declare that I have carefully read and fully understood the notes on use, notes on hygiene, warnings and other declarations in relation to the **MAM PREEMIE Size 3**, and that I agree with them.

I am aware that the MAM PREEMIE Size 3 does not conform to the European Safety Standard EN 1400 as the extra small teat cannot satisfy the biting and pulling tests which are required for older children with teeth. The MAM PREEMIE Size 3 may therefore only be used for premature babies with a weight of up to 2,500 g. I am therefore responsible for ensuring that the MAM PREEMIE Size 3 is kept out of the reach of older children with teeth and children who weigh more than 2,500 g and cannot be used by these children.

I am aware that the pacifier may only be used under strict compliance with the notes which were mentioned at the start. I myself am responsible for ensuring that this happens. If the pacifier is used in an inappropriate or incorrect way, MAM shall not be liable in any way for any damages or detriments whatsoever.

.....
Place, date

Page 1 of 4

.....
Signature of parent or legal guardian

MAM Babyartikel GmbH

Lorenz-Mandl-Gasse 50, 1160 Vienna, Austria
Phone +43 1 49 141 0, Fax +43 1 49 141 404
E-mail office@mambaby.com, Internet mambaby.com

Bank affiliation: Bank Austria, Sort code 12000, A/C 255-106-002/00
BIC BKAUATWW, IBAN AT67120002551060 0200
FN 10465Z, Vienna Commercial Court

MAM PREEMIE SIZE 3



MAM PREEMIE Size 3 is a pacifier which was developed specifically for premature babies and, according to Directive 93/42/EEC Class I, is classified as a medical product. The teat of the MAM PREEMIE Size 3 is specially designed to cater for premature babies - it is naturally smaller, softer and therefore not as firm to the bite as regular pacifiers which are used for older children with teeth and children who weigh more than 2,500 g.

For this reason, the MAM PREEMIE Size 3 does not meet the strict European Safety Standard which applies to pacifiers EN 1400 (Point 6.2.3.1. Tear resistance of the teat and Point 6.2.2.1. Puncture resistance of the pacifier are not quite met). However, the MAM PREEMIE Size 3 does satisfy the American Safety Standard for pacifiers (16 CFR 1511).

The MAM PREEMIE Size 3 may therefore only be used for premature babies with a weight of up to 2,500 g. It must be kept out of the reach of older children with teeth and children who weigh more than 2,500 g.

Special notes on use and hygiene must be followed when using the MAM PREEMIE Size 3:

Notes on use:

- MAM PREEMIE Size 3 helps a child to make the transition to a regular pacifier. It may only be used with children who weigh less than 2,500 g. As soon as your child reaches a weight of 2,500 g, please switch to a regular newborn pacifier (e.g. MAM Start).
- The teat is asymmetrical to ensure that it is easier to switch from Size 1 and 2. Always place the pacifier in the mouth with the flat side of the pacifier pointing down. As soon as your child learns how to rotate the pacifier in their mouth, we recommend switching to a pacifier with a symmetrical teat, e.g. MAM Start. This ensures that your baby can never put the pacifier in their mouth the wrong way round.
- Do not dip the teat in sweet substances or medicines. This could cause decay.
- Do not panic if the pacifier ends up fully inside your child's mouth. The design of the pacifier means that the child cannot swallow it. Carefully remove it from the mouth.
- The pacifier should be replaced at the latest after 1-2 months of use.
- MAM recommends that you wean your child off a pacifier by the time the child is 36 months old!



Notes on hygiene:

Sterilise the pacifier before you first use it and between each use. Place the pacifier in boiling water for 5 minutes, allow it to cool and squeeze the water out of the teat.

For the safety of the child!

Warnings:

- Never attach ribbons or cords to a pacifier. Your child could strangle themselves on them.
- Examine the pacifier before each use, especially if your child has teeth. Pull the pacifier in all directions. Please note that silicone is less resilient to biting than latex. Pull the pacifier in all directions, and replace it immediately on the first sign of any damage or material weakness.
- Never place a pacifier in direct sunlight or heat and never leave it in disinfectant solutions ("sterilising solution") for longer than recommended as this can have an adverse effect on the teat.

MAM PREEMIE Size 3 may only be used for premature babies with a weight of up to 2,500 g. Please keep the MAM PREEMIE Size 3 out of the reach of older children with teeth and children who weigh more than 2,500 g and make sure that it cannot be used by these children. The teat could be bitten off by children with teeth!